Anti-HPV bioprotein gel

[Product Name] Anti-HPV Bioprotein Gel

[Model Specifications] g/tube

[Product Structure] This product consists of a gel and a vaginal applicator. The gel is composed of anhydrided bovine β -lactoglobulin, purified water, carbomer, and menthol. The vaginal applicator consists of a cap, tube, push rod, and sealing plug, all made of PET.

[Key Performance Indicators]

- 1. The cytotoxicity of the materials used in this product should be no greater than Class II.
- 2. The product should not cause allergic reactions or intradermal irritation.

[Scope of Application] Used to reduce local HPV load

Contraindications Contraindicated for those allergic to this product, menstruating women, and pregnant women.

[Instructions]

- 1. Clean hands and vulva before use.
- 2. Open the package, remove the vaginal applicator, first remove the sealing plug, then remove the cap, and use the cap to push the push rod.
- 3. Lie supine with your hips elevated and legs apart. Gently push the cap to insert the gel deep into the vagina.
- 4. Use once daily, one tube at a time.

[Precautions]

- 1. Discontinue use during menstruation. Use with caution if you have skin allergies.
- 2. If you experience any burning sensation, redness, or swelling during use, discontinue use, wash thoroughly, and consult a physician if necessary.
- 3. This product is recommended for use alone. If combined with other medications, consult a physician.
- 4. This product is disposable and should not be used if the aluminum foil bag is damaged or expired.
- 5. Urinate before use to avoid affecting the effectiveness.
- 6. This product is for external use only. Do not take orally. Keep out of reach of children.
- 7. Do not use for the prevention of sexually transmitted diseases during intercourse.



One-time use, destroyed after use



Do not use if the packaging is damaged

[Storage Conditions and Methods] Store in sealed packaging in a well-ventilated, dry, light-proof, and free of corrosive gases in a room with a relative humidity not exceeding 80%.

【Registrant/Manufacturer】 Changde Yinuo Biopharmaceutical Co., Ltd.

【 Registrant/Manufacturer Address/Manufacturer Address 】 Fengkang Avenue,

Xicheng Community, Dajinggang Town, Anxiang County, Changde City, Hunan Province (within Jinggang Park)

【After-Sales Service Provider】 Changde Yinuo Biopharmaceutical Co., Ltd.

Transport of Schools Production License Number Transport of SchoolsThe production License No. 20210068

The production License No. 20210068

【Registration Certificate Number/Product Technical Requirements Number】
Medical Device Registration Approval No. 20212181962

[Manufacture Date] See outer box for details

[Product Expiration Date] 2 years

Note: Products without the traceability code of Changde Yinuo Bio-Pharmaceutical Co., Ltd. are all counterfeit and inferior products.