
Bioprotein Oral Gel Instructions

【Product Name】 Bioprotein Oral Gel

【Model Specifications】 g/tube

【Product Structure】 This product consists of a gel and a container. The gel is composed of mussel mucin, hydroxyethyl cellulose, propylene glycol, glycerin, and purified water.

【Key Performance Indicators】

1. The cytotoxicity of the materials used in the product should be no greater than Class II.
2. The product should be non-allergenic and non-irritating to the mouth.

【Scope of Application】 Used to relieve pain caused by oral ulcers, oral inflammation, denture wounds, or surgical wounds.

【Contraindications】 Contraindicated for those allergic to this product.

【Instructions】 Brush your teeth 2–3 times daily with Bioprotein Oral Gel, brushing for 1–3 minutes each time. If the oral mucosa is damaged, rinse the foam in your mouth for 2–3 minutes after brushing.

【Precautions】

1. No known problems have been reported with use during pregnancy and breastfeeding.
2. If a burning sensation or allergic reaction occurs during use, discontinue use immediately and seek medical attention if necessary.
3. Do not use after the expiration date. Once opened, use within 3 months. If not used for an extended period, refrigerate.

【Storage Conditions and Instructions】 Keep out of reach of children. Store the container and outer box in a well-ventilated, dry, light-proof room with a relative humidity not exceeding 80%, free from corrosive gases.



Do not use if the packaging is damaged

【Registrant/Manufacturer】 Changde Yinuo Biopharmaceutical Co., Ltd.

【Registrant/Manufacturer Address/Manufacturing Address】 Fengkang Avenue, Xicheng Community, Dajinggang Town, Anxiang County, Changde City, Hunan Province (within Jinggang Park)

【After-Sales Service】 Changde Yinuo Biopharmaceutical Co., Ltd.

【Production License Number】 Hunan Food and Drug Administration Medical Device Production Permit No. 20210068

【 Registration Certificate Number/Product Technical Requirements Number 】

Hunan Medical Device Registration Permit No. 20212171960

【 Manufacturing Date 】 See outer box for details

【 Product Expiration Date 】 2 years

【 Product Instruction Manual Revision Date 】 April 25, 2021

Note: Products without the Changde Yinuo Biopharmaceutica Co., Ltd. traceability code are counterfeit and substandard products.