

Far Infrared Therapy Patch Instructions

【Product Name】 Far Infrared Therapy Patch

【Models and Specifications】 Product sizes include 50*80mm, 60*100mm, 110*140mm, 150*150mm, 150*180mm, 150*200mm, 200*200mm, 200*260mm, and 260*300mm, with a tolerance of $\pm 10\%$.

【Product Structure】 The far-infrared therapy patch consists of far-infrared ceramic powder, medical pressure-sensitive adhesive (made of acrylic resin), medical non-woven fabric, and silicone oil paper.

【Key Performance Indicators】

1. The cytotoxicity of the materials used in the product should not exceed Level II;
2. The product should not cause allergic reactions or intradermal irritation;
3. The surface of the product's hard paste (far-infrared ceramic powder) should be able to emit far-infrared radiation (biowaves) with a wavelength of $4\mu\text{m}$ - $16\mu\text{m}$, and its normal emissivity should be no less than 0.8.

【Scope of Application】 It promotes local blood circulation, assists in reducing inflammation, swelling, and pain. It is used as an adjunctive treatment for pain in the neck, shoulders, waist, legs, and joints.

【Contraindications】

1. Do not use on damaged skin or those with allergies.
2. Do not use on patients with high fever or pregnant women.

【Instructions】

Cleanse the affected area, remove the anti-adhesion barrier, and apply the therapeutic plaster to the skin surface, aligning it with the acupuncture points. Apply the plaster simultaneously to one or more effective acupuncture points as needed.

【Precautions】

1. Do not use after the expiration date.
2. Carefully inspect the inner packaging before use. Do not use if it is damaged or contaminated.
3. This product is for single use only. Reuse is strictly prohibited. Dispose of after use.
4. Please read the instructions carefully before use.



One-time use, destroyed after use



Do not use if the packaging is damaged

【Storage Conditions and Methods】 This product should be stored indoors with a relative humidity not exceeding 80%, away from light, free of corrosive gases, and good ventilation.

【Registrant/Manufacturer】 Changde Yinuo Biopharmaceutical Co., Ltd.

【Registrant/Manufacturer Address/Manufacturing Address】 Fengkang Avenue,

Xicheng Community, Dajinggang Town, Anxiang County, Changde City, Hunan Province (within Jinggang Park)

【After-Sales Service】 Changde Yinuo Biopharmaceutical Co., Ltd.

【Production License Number】 Hunan Food and Drug Administration Medical Device Production Permit No. 20210068

【Registration Certificate Number/Product Technical Requirements Number】
Hunan Medical Device Registration Permit No. 20212091713

【Manufacture Date】 See outer box for details

【Expiration Period】 2 years

【Product Instruction Manual Revision Date】 April 25, 2021

Note: Products without the Changde Yinuo Biopharmaceutical Co., Ltd. traceability code are counterfeit and substandard products.