

Sterile Wound Dressing Instructions

【Product Name】 Sterile Wound Dressing

【Model Specifications】 mL/bottle

【Product Performance, Structure, and Composition】 The sterile wound dressing consists of a dressing and a measuring bottle. The dressing is composed of purified water, carbomer, and hydroxyethylcellulose. It is sterilized by irradiation and supplied sterile.

【Key Performance Indicators】

1. The cytotoxicity of the materials used in the product should not exceed Class II;
2. The product should not have delayed-type hypersensitivity reactions or intradermal irritation;
3. Sterility: Sterile after irradiation sterilization.

【Scope of Application】 Intended for the care of minor wounds, abrasions, cuts, and other non-chronic wounds, as well as the surrounding skin.

【Contraindications】

1. Use with caution in those with allergies;
2. Contraindicated in pregnant or lactating women;
3. Use with caution in children.

【Directions】 Cleanse the skin thoroughly and apply the product to the skin.

【Precautions】

1. Do not use this product after the sterilization expiration date.
2. This product is sterile. Please carefully inspect the inner packaging before use. Do not use if it is damaged or contaminated.
3. This product is for single use only. Reuse is strictly prohibited. Please dispose of it after use.
4. This product is for external use only. Please strictly follow the instructions for use. Injection and other illegal operations are prohibited.
5. Please read the instructions carefully before use.



One-time use, destroyed after use



Do not use if the packaging is damaged



Sterilized by irradiation, for external use

Storage Conditions and Methods: Store indoors in a ventilated, dry, light-proof environment with a relative humidity not exceeding 80%. Store in a well-ventilated, dry environment free from corrosive gases.

[Registrant/Manufacturer] Changde Yinuo Biopharmaceutical Co., Ltd.

[Registrant/Manufacturer Address/Manufacturing Address] Fengkang Avenue,
Xicheng Community, Dajinggang Town, Anxiang County, Changde City, Hunan Province (within
Jinggang Park)

[After-Sales Service] Changde Yinuo Biopharmaceutical Co., Ltd.

[Production License Number] Hunan Food and Drug Administration Medical Device
Production Permit No. 20210068

[Registration Certificate Number/Product Technical Requirements Number]

Hunan Medical Device Registration Permit No. 20212141546

[Manufacturing Date] See outer box for details

[Product Expiration Date] 2 years

[Product Instruction Manual Revision Date] April 25, 2021